



MT10 - Screening Tympanometer

# Inclusions and Install Guide

## **Inclusions with your device:**

- MT10 device with handle
- Set of tympanometry ear tips
- Batteries, 4 x AA
- Cleaning accessories
- Calibration certificates & Warranty Card
- Instructions for Use (or available via website)

## **Additional accessories if purchased:**

- 4 in 1 test cavity assembly
- Carrying case
- Portable thermal printer & 2 rolls of paper
- Diagnostic Suite software on USB
- OtoAccess® Database on USB
- Infra-red USB Adapter
- Additional probe tip
- Additional sets of ear tips

## **First use of your MT10 device:**

1. Place the batteries into the rear of the unit, ensuring they are placed in the correct orientation. Replace the battery cover.
2. Press the on/off switch, the MT10 will power on and the menu displays on the screen.
3. Select 'New Test'
4. Attach appropriate sized tip to the probe end.
5. Conduct your measurement in the ear, the result will be displayed on the screen.
6. You can save the results or go on to test the other ear.

This quick guide does not replace the instructions for use. Please visit our support page on the global Interacoustics website for more support and to download the instructions for use:

<https://www.interacoustics.com/tympanometers/mt10/support>

Our MT10e (extended) allows for ipsi reflex testing, if you wish to turn this off, then do so through the Settings and set as 'never measure'.

It is important to correctly clean your device to ensure good working order. Please clean your instrument daily to keep the probe clean and free from debris. Instructions on how to do this are located in the manual.

If you have software with your device, please refer to the quick install guide on this in separate documents; Otoaccess, Diagnostic Suite or Titan Suite.